



## **Pango Activity Chair Accessories**

Headrest (PA2100)

Laterals (PA2110)

Stabilizer & Rocker (PA2301, PA 2302)

Wheels Kit for Wheeled Mobility Base (PA2310)

Trunk Harness (PA2201, PA2202)

Hip Guides (PA2211)

Abductor (PA2212)

Tray (PA2213)

Footrest (PA2401, PA2402)

Foot and Ankle Positioners (PA2410)

Guide Bar (PA2501, PA2502)

## Headrest (PA2100)

Adjustable height, depth, and angle for posterior head and neck support.

1. Unscrew the four top height screws in the middle of the chair back.
2. Attach the headrest using those four screws.
3. Adjust the headrest into desired position and tighten.



*Figure 1: Headrest*

## Laterals (PA2110)

Height and width adjustable to stabilize and align child's trunk.

1. Unscrew the two middle height screws in the chair back.
2. Insert laterals into the slit in the side of the chair back.
3. Screw the two middle height screws back into the chair back.
4. Adjust each lateral using the two screws and slot.



*Figure 2: Laterals*

### Stabilizer & Rocker (PA2301, PA 2302)

Stabilizer anchors the chair, preventing it from moving. Easily converts to Rocker, encouraging dynamic movement.

1. Attach the rails to the chair legs in stabilizer or rocker orientation using the four long screws.
2. Attach the cross brace using the two short screws.

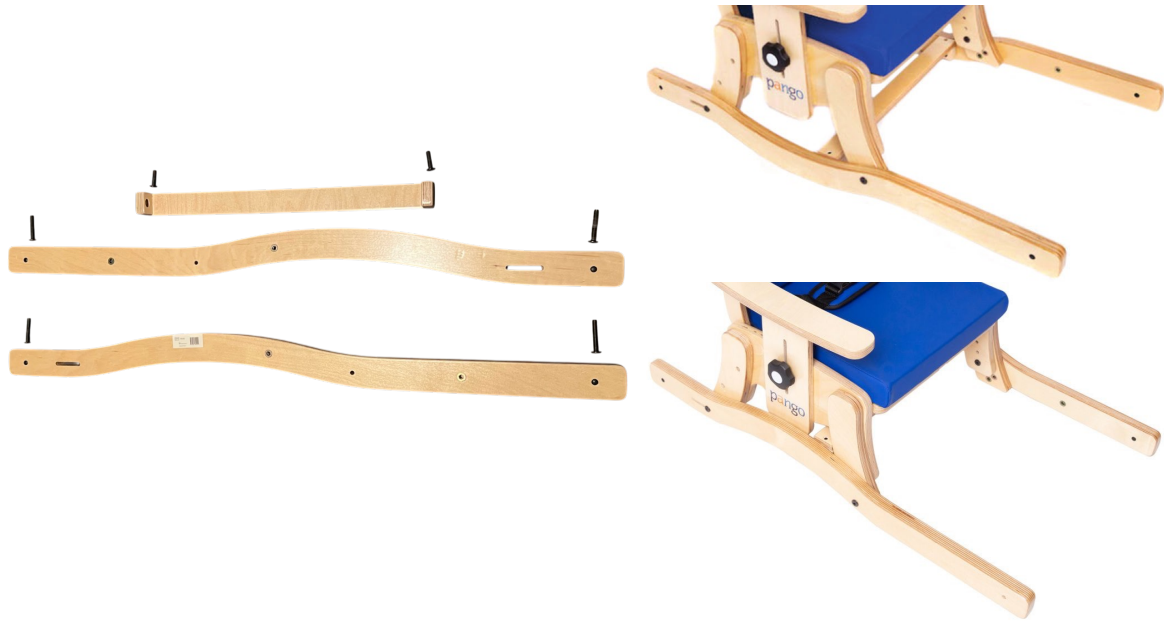


Figure 3: Stabilizer & Rocker

### Wheels Kit for Wheeled Mobility Base (PA2310)

Four casters convert Stabilizer into a Wheeled Mobility Base

1. Remove all four hole-plugs.
2. Attach all four wheels using the four locknuts and bolts.
3. Add the four locknut covers.



Figure 4: Wheels Kit for Wheeled Mobility Base

## Trunk Harness (PA2201, PA2202)

Provides child with anterior trunk support.

1. Attach the four strap anchors to the back of the chair using the screws already in the back.
2. Insert the four trunk harness straps into the anchors and close.
3. Adjust the straps for use.



*Figure 5: Trunk Harness*

## Hip Guides (PA2211)

Provides hip and thigh support.

1. Determine placement of hip guides.
2. Remove backing on adhesive and place.
3. Hip guides can now be removed and reattached using hook and loop fasteners.



*Figure 6: Hip Guides*

## Abductor (PA2212)

Limits forward sliding, prevents legs from scissoring, is depth adjustable.

1. Attach abductor at desired depth using the one screw.



Figure 7: Abductor

## Tray (PA2213)

Tray fits both size chairs.

1. Attach both brackets to the tray using two screws each.
2. Screw the spring pins into the brackets and adjust the nut on each to set the correct depth.
3. Attach the tray by pulling the spring pins, sliding it into place on the arm rests, and releasing.
4. Always make sure the spring pins lock the tray into place before use.
5. Adjust or remove the tray by pulling the spring pins and moving.



Figure 8: Tray



### Footrest (PA2401, PA2402)

Height and angle adjustable.

1. Loosen the four hand knobs.
2. Adjust to desired height and angle.
3. Lock in place by tightening the four hand knobs.
4. Attach the footrest to the front legs of the chair using the two smaller screws.
5. If in use, attach the footrest to the stabilizer/rocker using the two longer screws.



Figure 9: Footrest

### Foot and Ankle Positioners (PA2410)

Adjustable heel and toe loops.

1. Strap heel loops onto footrest plate.
2. Strap toe loops onto footrest plate.
3. Adjust heel loops to desired position and tightness.
4. Adjust toe loops to desired position and tightness.



Figure 10: Foot and Ankle Positioners

## Guide Bar (PA2501, PA2502)

Bar is height adjustable.

1. Attach the guide bar with two screws on each side.
2. Adjust height by pushing in both buttons and pulling in desired direction.



Figure 11: Guide Bar

**See Pango User Manual for important Safety, Care and Maintenance, and Warranty information.**